

TableEDU 11 - Offline

Anatomage

Create your own Offline Installer USB



Thank you for requesting your Table 11 software update!

The following instructions provide an alternative to the Table 11 Online Installer. This alternative lets you skip the wait for an offline USB drive. You can create your own offline USB installer to use at the Table.

What does this process involve?


The Google Drive installation process involves three main steps:

1. Installing the Google Drive for desktop application on your computer. This will facilitate a faster file transfer compared to downloading directly from the Google Drive website.
2. Transferring the TableEDU 11 installer files from Google Drive to the Table using a portable drive. **Your drive must be able to store 315 GB.**
3. Running the TableEDU 11 offline installer on the Table.


Table 11 - Create your own Offline Installer USB

1. Add the TableEDU 11 Installer Files to your Google Drive

- On a web browser, sign into your Google account.
- Navigate to https://drive.google.com/drive/folders/1zO6_rwd9lelqXlptYeINWCBPH0rTFds?usp=sharing
- On the left hand side, select "Shared with me". Right-click the "TableEDU 11.0 Full USB Installer" folder to view more options. Select "Organize" then select "Add shortcut".

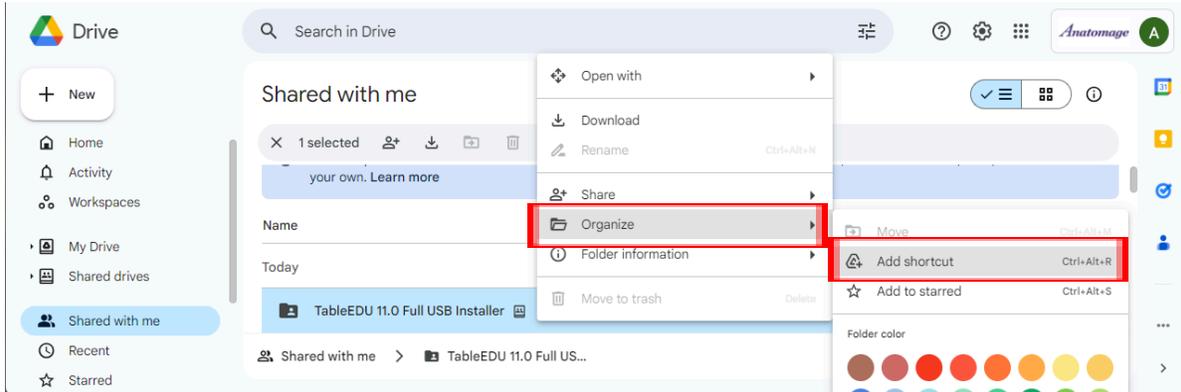

- Select "My Drive" then select "Add" to add the shortcut to your Drive.

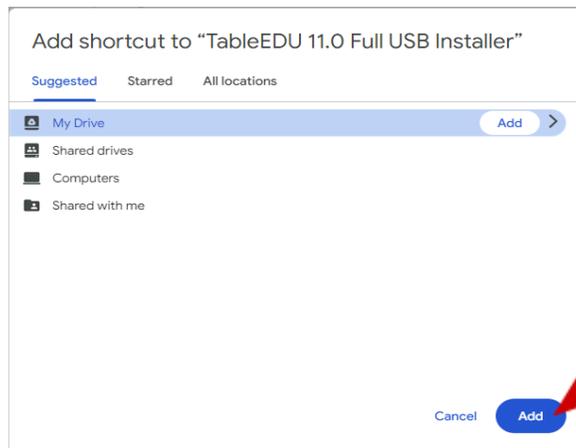


2. Install Google Drive for Desktop onto your computer (not the Table PC)

- Navigate to <https://www.google.com/drive/download/>
- Download and install Drive for desktop onto your PC, and connect it to your Google account.

Safely store your files and access them from any device


Choose folders on your computer to sync with Google Drive or backup to Google Photos, and access all of your content directly from your PC or Mac

[Download Drive for desktop](#)

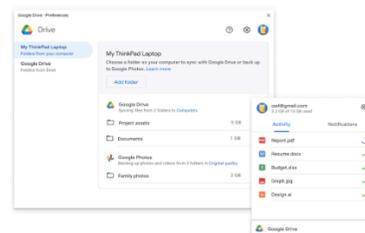


3. Move the TableEDU 11 Installer Files to the Table

- a. In Google Drive for desktop, navigate to “My Drive” to find the “TableEDU 11.0 Full USB Installer” folder shortcut.

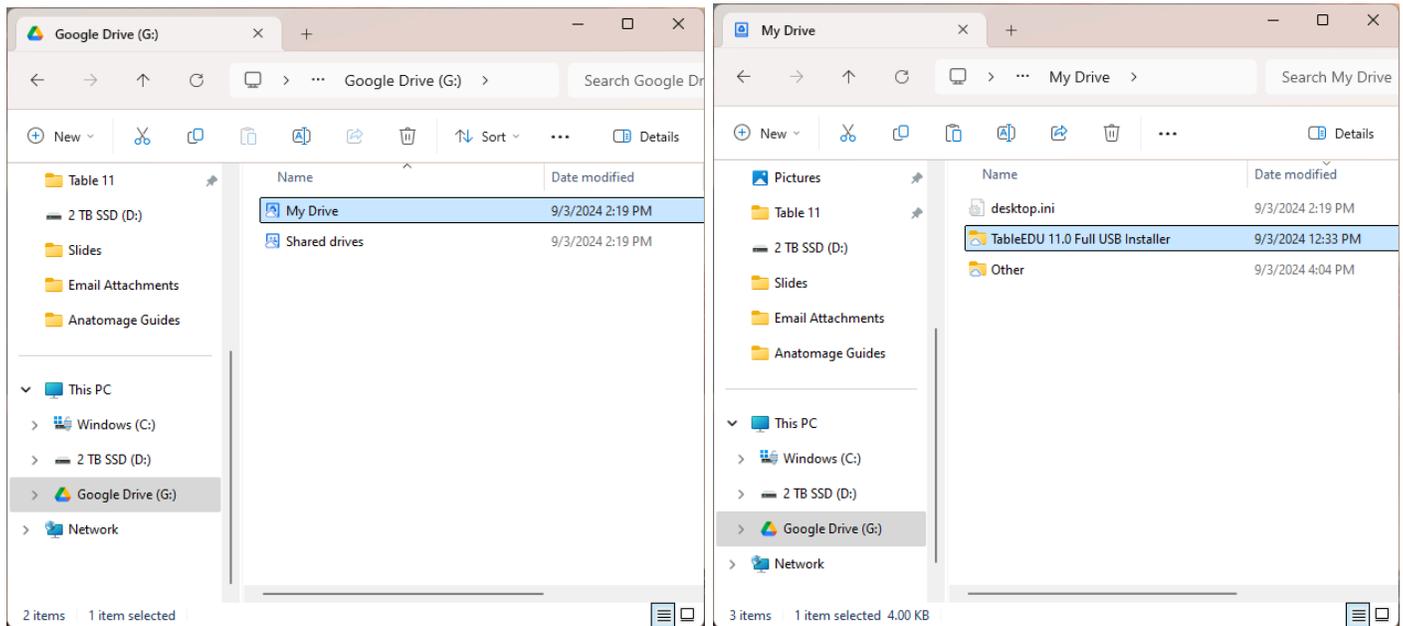


- b. Open the “TableEDU 11.0 Full USB Installer” folder and copy all TableEDU 11 Installer files to a USB thumb drive or SSD. **The USB thumb drive or SSD must be able to hold 315 GB of data.** This may take a long period of time to complete.

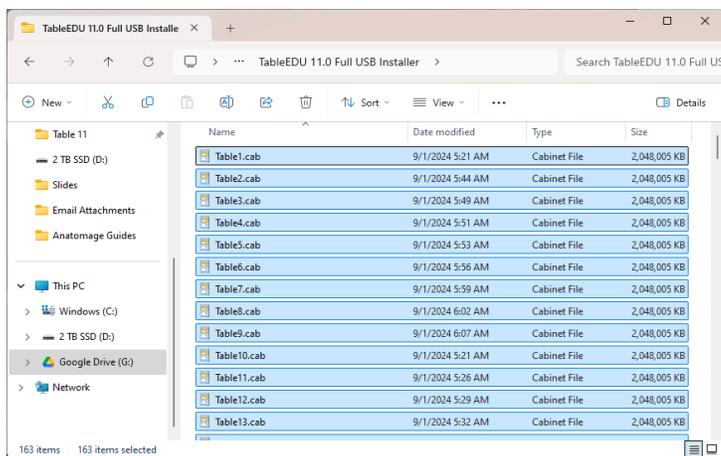

- c. Plug your TableEDU 11 Installer thumb drive into the Anatomage Table USB port.


4. Run the TableEDU 11 Installer

- a. In the Windows File Explorer dialog, locate the TableEDU_Installer_USB.exe file, and double click it to open the installer.

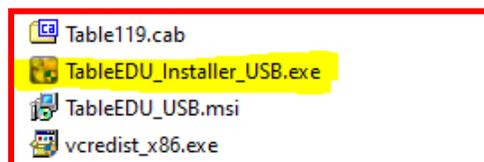


b. A TableEDU software activation dialog will pop up.

Press the “Advanced” button.

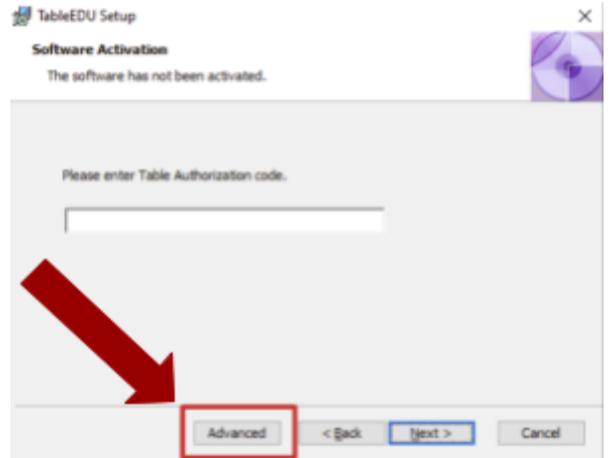

c. In the following dialog, enter the License Code provided to you by Anatomage where prompted. Include all dashes and capital letters. Then press “Next”.

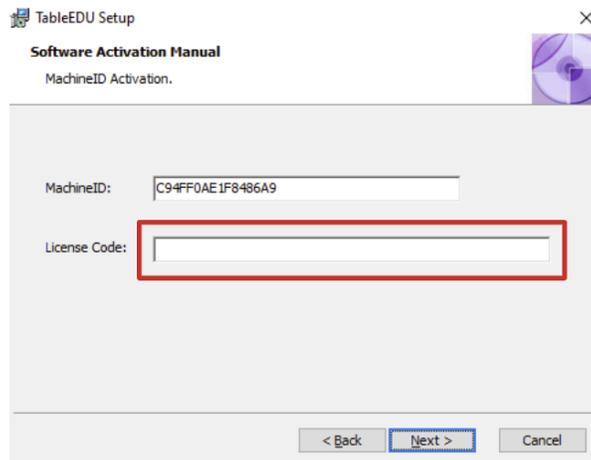

d. Confirm the Installation Folder by tapping “Next”.

a. The installation will begin. Installation process may take a few hours.

i. **The USB drive must remain plugged in for the entire installation duration.**

If you have any error messages or issues with this process, please refer to the FAQ document sent to you by Anatomage. If the FAQ document does not resolve your problem, contact the team member who sent you the update codes or training@anatomage.com.

IMPORTANT: DO NOT SHUT DOWN TABLE WHILE INSTALLATION IS RUNNING

4. Post-Installation

a. After the installation is complete, please verify the new software is opening correctly and confirm this with us via email. **Please email the team member who sent you the update code or training@anatomage.com to confirm the update is complete and successful.** Upon confirmation, we will send you the updated resources for the new Table 11 software for you to review (What’s New Information, Interface Guide, Manuals, etc).